




L'Atelier

MOUNTING & UN-MOUNTING OF 3D PRINT FRAMES

Lens

1. Always remove demo lens inside out **without heating**
2. To cut the lens, **always use the demo lens**: the top of the bevel (red line on second picture). **Scan the demo lens** and do not increase or reduce its size
3. To insert the prescription lens: **without heating the front**, always from the inside – first start with the groove nasal side
4. Clip the prescription lens inside the groove **without heating the front**

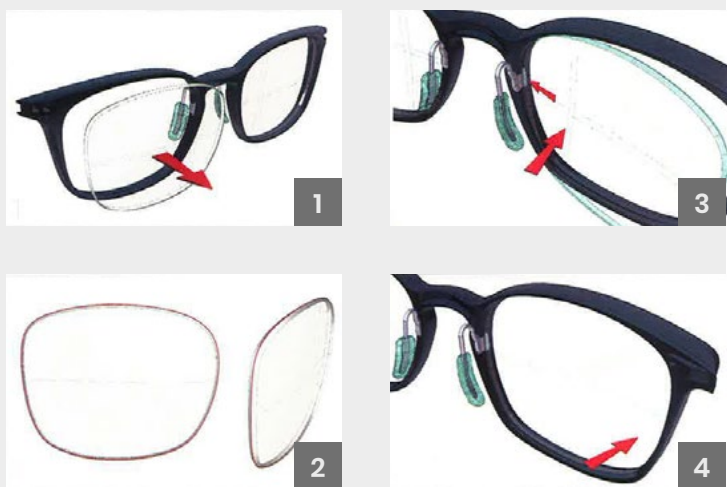

LINOTEC pad-arm

1. Always remove demo lens inside out **without heating**
2. **Always** remove the LINOTEC pad-arm **by the back**
3. To change the nose pad, remove it downward
4. To put the LINOTEC pad-arm, the lens must be removed. Introduce the pad-arm by the back and slide in the notch

If you have any questions, please contact us!
info@mag-intl.com or 450-986-4178



MAG-INTL.COM



FOLLOW US ON FACEBOOK + INSTAGRAM